3mach®-Filterzylinder (patentiertes Produkt)

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3mach® filter cylinders (patented product)

Use: Accessories for sieve baskets and RDGs in the field of instrument reprocessing

The 3mach® filter cylinder consists of two parts and can contain one, two or three filter discs and be combined with a holding device. The filter discs are separable and reusable. To clean, please separate the individual filter discs and clean them from the dirty side (side without logo). Cleaning is carried out with a cleaning gun under running water; a fine brush or an ultrasonic bath can also be used.

filter cylinder

with thread

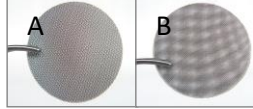
with tube connection/holding device

for flushing manifold

filter disc set

Caution: All 3mach® products must be cleaned before first use. Ensure that the filters in the 3mach® filter cylinder are clean, not bent, and correctly inserted in the correct orientation C (logo at the top). Bent discs can be smoothed out with a flat object. The filter grids may be arranged differently on top of each other (visible on grid pattern A or B). The colored outer silicone ring indicates that one filter disc (blue ring) or two filter discs (red ring) are included. Open Luer-Lock or other openings on the cylinder must be closed. 3mach® flushing and filter cylinders must not be used on humans.

Preparation: Suitable for alkaline cleaning processes in the areas of machine cleaning, thermal disinfection, and drying (up to 120 °C). The cylinders and filter discs can be sterilized.

Liability: 3mach GmbH bears no liability for injury or damage resulting from the use of 3mach® products. This also applies to individually converted and modified 3mach® products and their misuse.

Manufacturer: Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany
Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, www.3mach.de

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Hersteller: Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany