## 3mach® K-System

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Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, www.3mach.de

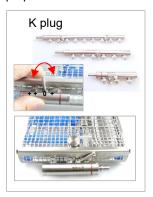
G-44 Gebrauchsanweisung Version 11/2025 3mach® K-System web: www.3mach.de

3mach® copyright (disclosure of information is prohibited)

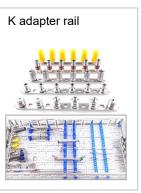
## 3mach® K-system

Use: 3mach® K-system in the area of instrument preparation.

The 3mach® K-system consists of pluggable K adapters or a K adapter rail that are used in conjunction with a matching K plug. K adapters can be attached to a 3mach® basket wall or bottom, or to a retaining rail or bracket. The K-system can have 2–9 flushing connections that can be changed via a quick coupling and connected to a K plug. This enables standardized instrument validation of cavity products, making the system particularly suitable for setting up instrument sets used in surgery.

CE

Caution: All 3mach® products must be cleaned prior to initial use. The silicone disc seals are not permanently fixed and any missing silicone discs must be replaced. Bent fixing parts or plug adapters must be repaired. Ensure that the quick-fixing device tightens the K plug and creates a seal. The K plug must be inserted in position 0 straight and without resistance. By turning the locking lever to the left or right, the locking strength + / ++ is determined. The K6 and K9 plugs tend to bend the outer adapters if they are not inserted straight (caution: lever arm effect). The K9 plug must therefore be used with both hands. It is recommended to close any K adapters that are not needed or to replace them with a K closure adapter. 3mach® K-systems must not be used on humans. We recommend removing K-connectors from the sieve basket after cleaning and regularly cleaning the filters they contain (indicated by a red ring)

Preparation: Suitable for alkaline cleaning processes in the area of mechanical cleaning, thermal disinfection and drying

(120 °C) and use in medical technology during fractionated vacuum processes (134 °C). For further instructions for

use or types of use, please contact us by email at info@3mach.de.

Liability: 3mach GmbH bears no liability for injury or damage resulting from the use of 3mach® products. This also applies to

individually converted and modified 3mach® products and their misuse.

Manufacturer: Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany

Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, www.3mach.de

(

web: www.3mach.de

G-44 instruction for use

Version 11/2025

3mach® K-system

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**Hersteller:** Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany