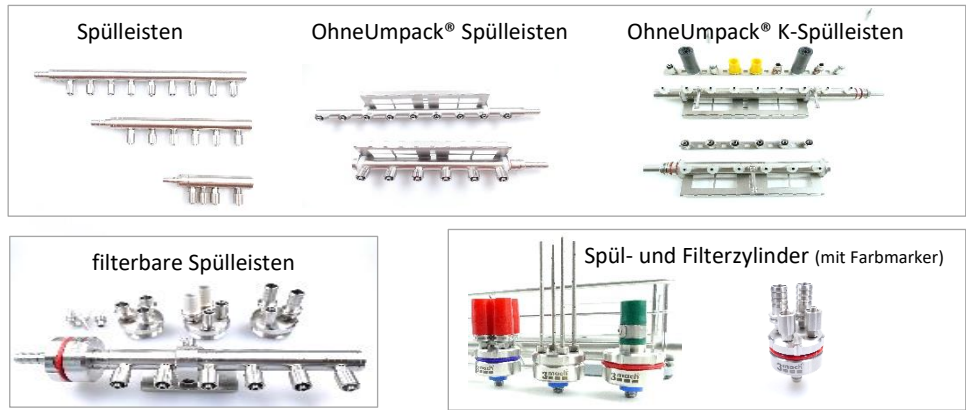
## 3mach®- Medienverteiler

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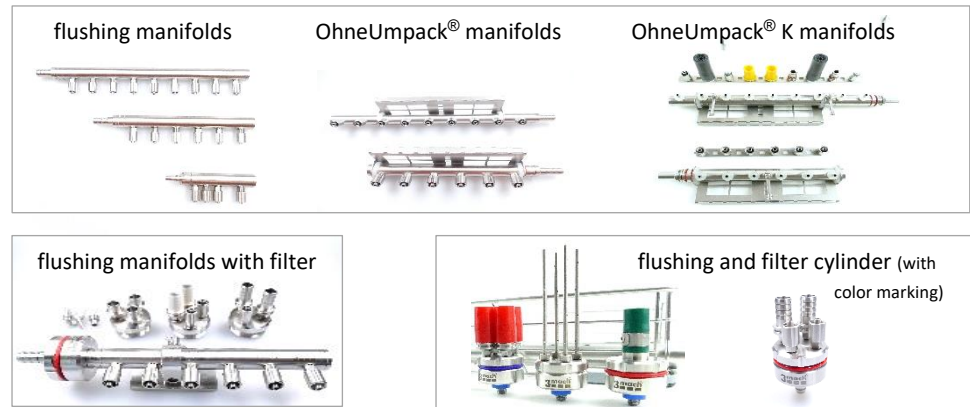
## 3mach® manifolds of flushing supply

**Use:** Manifolds of flushing supply in the area of instrument preparation.

3mach®-manifolds of flushing supply can be connected to a filter cylinder, a mounting block and/or a suspension device from 3mach company. 3mach®-manifolds of flushing supply are modular in design and can be adapted to the cavity instruments to be cleaned using adapter rails and enables the 3mach OhneUmpack®-System. The 3mach® flushing and filter cylinder consists of two parts and may contain one, two or three filter discs. The filter discs can be separated and used multiple times. Filter discs must be cleaned regularly (see 3mach® filter cleaning).

**Caution:** All 3mach® manifolds of flushing supply must be cleaned prior to initial use. Never use flushing manifolds on people. The flushing instruments must be securely connected to the manifolds and be able to pass the flushing supply through them. Unused adapters should be sealed to reduce pressure loss. If the designated mounting parts on the manifolds are bent too often, the bending edge may tear in and break off. The cleaning instructions from the instrument manufacturer must be observed when using the manifolds of flushing supply.

**Preparation:** Suitable for alkaline cleaning processes in the area of mechanical cleaning, thermal disinfection and drying (up to 120 °C). For further instructions for use or types of use, please contact us by email at [info@3mach.de](mailto:info@3mach.de).

**Liability:** 3mach GmbH bears no liability for injury or damage resulting from the use of 3mach® products. This also applies to individually converted and modified 3mach® products and their misuse.

**Manufacturer:** Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany  
Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, [www.3mach.de](http://www.3mach.de)

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**Hersteller:** Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany