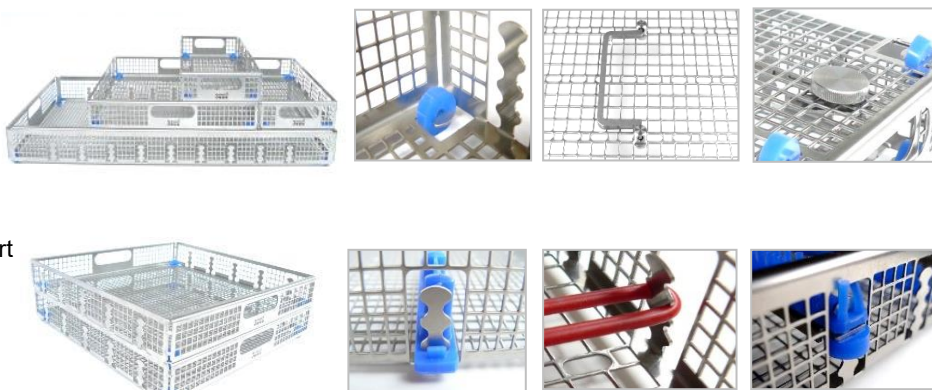
3mach® Multi-Systemkörbe

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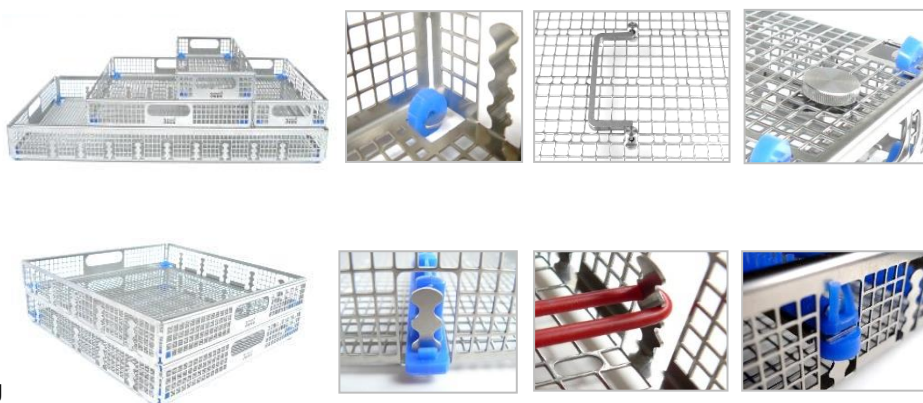
3mach® multi-system baskets

Use: 3mach® multi-system baskets in the area of instrument preparation.

3mach® multi-system baskets be connected together through stacking, placing over each other and joining and include changeable silicone buffer feet in different sizes. The side walls contain flexible moulding elements that can hold silicone bands, for example. The 3mach® multi-system baskets are also characterised by a functional grid. This improves the sieve basket drying process and retaining pins or pier pins can be functionally attached to this grid. The 3mach® multi-system basket has a large opening as a handrail and each a marking surface.

Caution: All 3mach® products must be cleaned prior to initial use. Defective or brittle silicone parts in the sieve baskets must be replaced. The side moulding elements can break off due to repeated strong bending (e.g. 45°). The silicone parts in the sieve basket should not be pulled too hard or removed and replaced too often, as this can cause them to tear. Never use 3mach® multi-system baskets on people.

Preparation: Suitable for alkaline cleaning processes in the area of mechanical cleaning, thermal disinfection and drying (120 °C) and use in medical technology during fractionated vacuum processes (134 °C). For further instructions for use or types of use, please contact us by email at info@3mach.de.

Liability: 3mach GmbH bears no liability for injury or damage resulting from the use of 3mach® products. This also applies to individually converted and modified 3mach® products and their misuse.

Manufacturer: Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany
Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, www.3mach.de

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Hersteller: Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany