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3mach® silicone sleeves and silicone caps without valve function

Use: Accessories for flushing modules and stainless steel adapters in the area of instrument preparation.

3mach® silicone sleeves and silicone caps without valve function made of coloured silicone are intended for transferring cleaning solutions in the cleaning process of a cleaning and disinfection machine. It must be ensured that the silicone sleeves and caps correspond to the connected adapter or flushing module internal diameters. The manufacturer's instructions and known standards for instrument reprocessing must also be observed.

Attention: All 3mach® silicone sleeves and silicone caps must be cleaned before first use and should not be stored openly. The cleaning solutions in cleaning and disinfection machine lead to embrittlement (hardening) of the silicone materials used. 3mach® silicone sleeve and silicone caps may not be used on humans.

- Note:** Silicone sleeves and silicone caps, which are used in instrument reprocessing and especially in the cleaning process, should be free of residual water after cleaning. Brittle or defective silicone sleeves and caps must be replaced.
- Preparation:** Suitable for alkaline cleaning processes in the area of mechanical cleaning, thermal disinfection and drying (up to 120 °C). For further instructions for use or types of use, please contact us by email at info@3mach.de.
- Liability:** 3mach GmbH bears no liability for injury or damage resulting from the use of 3mach® products. This also applies to individually converted and modified 3mach® products and their misuse.
- Manufacturer:** 3mach GmbH, Daimlerstrasse 25, 72414 Rangendingen, Germany
Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, www.3mach.de

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- Hersteller:** Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany