## 3mach®-Silikonringe

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## 3mach® silicone rings

**Use:** Accessories for 3mach® multi-system baskets and basket accessories in the area of instrument preparation.

3mach® silicone rings are accessory products for holding instruments during the preparation process. Ensure that silicone rings are not damaged by sharp instrument components and that these are only slightly tensioned between the spaced mounting elements (sieve basket moulded parts, stepped screws, etc.). Tensioned silicone rings will alter their shape irreversibly over time.

**Caution:** All 3mach® silicone rings must be cleaned prior to initial use and should not be stored in an exposed state. Cleaning solutions in cleaning and disinfection machines cause silicone rings to become hardened and brittle. Never use 3mach® silicone rings on people.

**Note:** Silicone rings used during instrument preparation and, especially, the cleaning process should be free of residual water following cleaning. Brittle or defective silicone rings on the adapters must be replaced.

**Preparation:** Suitable for alkaline cleaning processes in the area of mechanical cleaning, thermal disinfection and drying (up to 120 °C) and use in medical technology during fractionated vacuum processes (134 °C). For further instructions for use or types of use, please contact us by email at [info@3mach.de](mailto:info@3mach.de).

**Liability:** 3mach GmbH bears no liability for injury or damage resulting from the use of 3mach® products. This also applies to individually converted and modified 3mach® products and their misuse.

**Manufacturer:** 3mach GmbH, Daimlerstrasse 25, 72414 Rangendingen, Germany  
Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, [www.3mach.de](http://www.3mach.de)

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**Hersteller:** Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany