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3mach® silicone tubes and cords

Use: Accessories for flushing modules, stainless steel adapters and sieve baskets in the area of instrument preparation.

3mach® silicone tubes made of transparent silicone (inner diameter 1 – 10 mm with different wall thicknesses) are designed for conveying cleaning solutions during the cleaning process of a cleaning and disinfection machine. Ensure that the inner tube diameters correspond to the inner diameters of the connected adapters or flushing modules. Silicone cords can be used to secure instruments in sieve baskets. In addition, manufacturer specifications and recognised standards for instrument preparation must be observed in this context.

Caution: All 3mach® tubes and cords must be cleaned prior to initial use and should not be stored in an exposed state. The colour of silicone tubes changes from naturally transparent to light yellow following exposure to daylight over a longer period of time. Cleaning solutions in cleaning and disinfection machines cause the silicone materials used to become hardened and brittle. Never use 3mach® silicone products on people.

Note: Silicone tubes used during instrument preparation and especially the cleaning process should be free of residual water following cleaning. Brittle or defective silicone products must be replaced.

Preparation: Suitable for alkaline cleaning processes in the area of mechanical cleaning, thermodesinfection and drying (up to 120 °C). For further instructions for use or types of use, please contact us by email at info@3mach.de.

Liability: 3mach GmbH bears no liability for injury or damage resulting from the use of 3mach® products. This also applies to individually converted and modified 3mach® products and their misuse.

Manufacturer: 3mach GmbH, Daimlerstrasse 25, 72414 Rangendingen, Germany
Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, www.3mach.de

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Hersteller: Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany