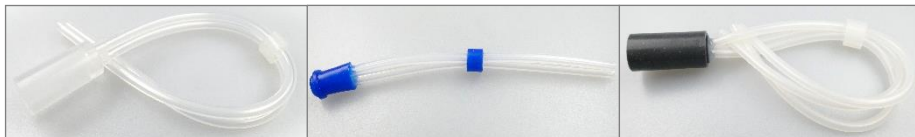
3mach®- Schlauchkappen

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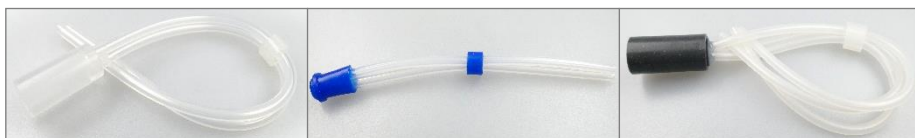
G45-Gebrauchsanweisung Version 02/2026 3mach®-Schlauchkappen web: www.3mach.de

3mach®- tube caps

Intended use: Accessories for cleaning machines in the field of instrument reprocessing.

3mach® tube caps with silicone tubes are available in different designs. The tube caps have two oder three silicone tubes and can be combined with 3mach®- LLW / LLM cap adapters or plug adapters for caps. The product is used with 3mach® rinsing cylinders or rinsing bars and increases the cost-effectiveness of rinsing fine-bore hollow instruments.

Caution:

3mach®- tube caps are wear parts and can become brittle more or less quickly depending on the type of application! Exposure to light causes the transparent silicone tubes to yellow. When handling, care must be taken not to pull on the tubes, as this can cause them to tear out. It is recommended to pull on the caps to disconnect the stainless steel adapter. 3mach®- tube caps with tubes must not be used on humans.

Note: 3mach®- tube caps are not suitable for hot air sterilization. Frequent cleaning and disinfection leads to embrittlement of the connections. Torn caps or tubing must be replaced.

Preparation: Suitable for alkaline cleaning processes in the area of mechanical cleaning, thermal disinfection and drying (up to 120 °C) and use in medical technology during fractionated vacuum processes (134 °C).

Liability: 3mach GmbH bears no liability for injury or damage resulting from the use of 3mach® products. This also applies to individually converted and modified 3mach® tube caps and their misuse.

Manufacturer: 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany
Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, www.3mach.de

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Hersteller: Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany