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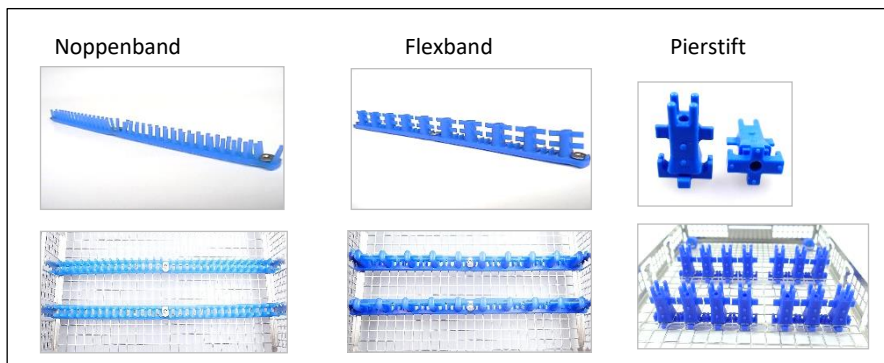
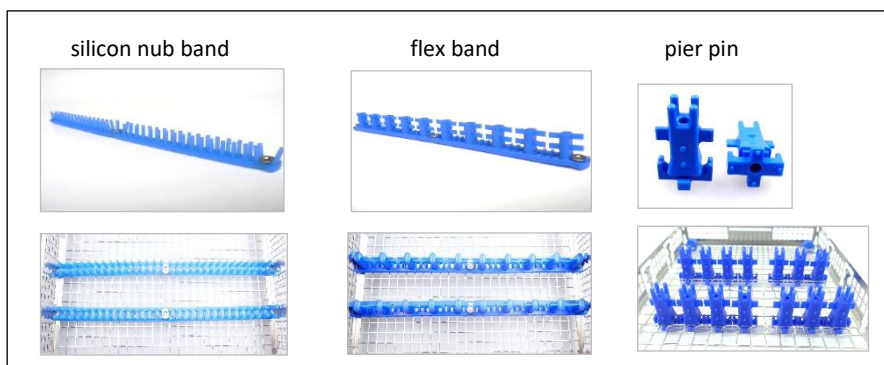
3mach® silicone holder accessories

Use: Accessories of flushing supply in the area of instrument preparation.

3mach® silicone holder accessories are made of high-quality silicone and are designed for use with 3mach® baskets. They are held in place by basket molded parts or in the base grid. Openings in the nubs and flex bands or a channel in the pier pins improve the cleaning performance and sterilizability of stored instruments. Nubs and/or retaining wings on the silicone pins enable multi-level storage of medical instruments with minimal rinsing shadows.

Caution: All 3mach® products must be cleaned before first use. Care must be taken to ensure that silicone accessories are not damaged by sharp instruments and objects. Torn silicone accessories must be replaced. Silicone accessories are not suitable for hot air sterilization. It is essential to ensure that the silicone bands between the molded parts are slightly tensioned and that the pier pins are placed in the base grid. No sharp-edged adapters, adapter threads, or instrument parts may be inserted into the pier channel. 3mach® silicone holder accessories must not be used on humans.

Preparation: Suitable for alkaline cleaning processes in the areas of machine cleaning, thermal disinfection, and drying (up to 120 °C), as well as for use in medical fractional vacuum processes (134 °C). For further instructions for use or types of use, please contact us by email at info@3mach.de.

Liability: 3mach GmbH bears no liability for injury or damage resulting from the use of 3mach® products. This also applies to individually converted and modified 3mach® products and their misuse.

Manufacturer: Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany
Tel.: 0049 (0) 74 71-98 41 18, Fax: 0049 (0) 74 71-98 41 19, www.3mach.de

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Hersteller: Firma 3mach GmbH, Daimlerstraße 25, 72414 Rangendingen, Germany